



12 in 1

LuxSPA Master

Multi-functional spa
facial skin care

Quickly treat skin, Safe, comfortable, effective

China Care Medical Equipment Co., Ltd
Address: Room B101, Zhongbang Business Building, Pu'ing
Road No. 87, Tianhe, Guangzhou,
China web: www.chinacaremedical.com



Hydra
dermabrasion



Diamond
dermabrasion



Bi-polar RF



Three polar RF



High
frequency



Skin
scrubber



Ice/hot
hammer



Oxygen
spray gun



Dr.pen



Plasma




Two BIO handle



Working Principle


1 HYDRA DERMABRASION WITH 8 TIPS

Effectively improve skin texture, eliminate wrinkles, and quickly improve skin texture, remove acne and blackheads quickly and easily without any pain or discomfort.

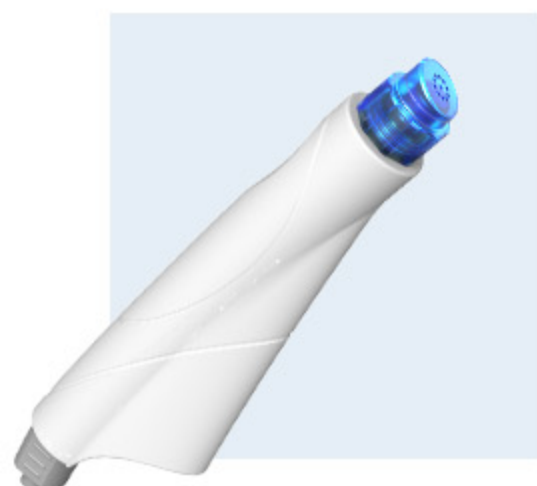



2 DIAMOND DERMABRASION WITH 9 TIPS

Diamond dermabrasion work head covering dead skin cells and absorbing them can completely purify keratin. Effectively promote blood flow to the skin and help produce collagen.

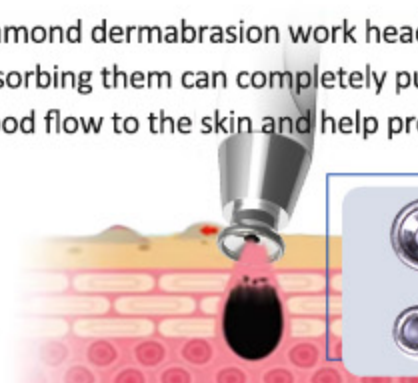

3 BI-POLAR RF

Heats skin tissue to absorb large amounts of heat and then smoothes it with new collagen. Wrinkles, tighten skin, delay skin aging, dilute fine lines, remove dark circles.

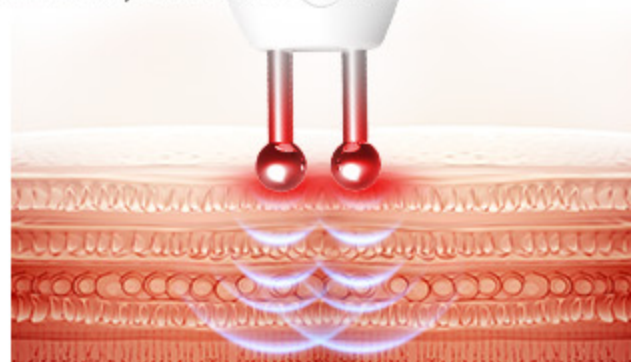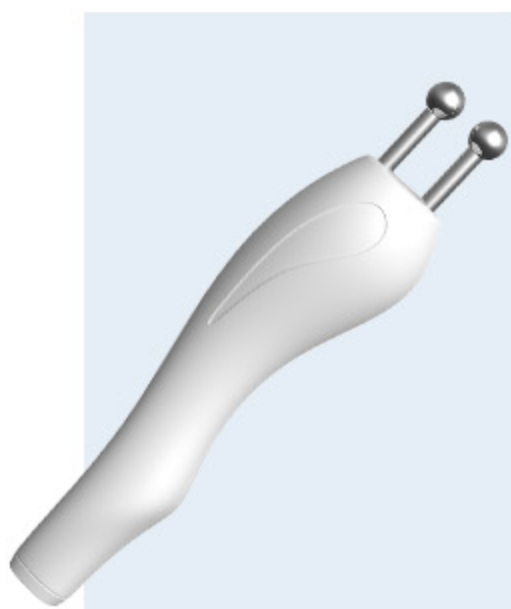



4 THREE POLAR RF

Radio frequency radio waves enter the deep part of the skin, and the internal natural resistance movement causes the skin to heat up. The use of dermal collagen cellulose immediately produces a contraction principle at 45-60C' which can smooth wrinkles, tighten the skin and delay skin aging.




5 OXYGEN SPRAY GUN

Nano oxygen gun (deep water replenishing) nano particles spray oxygen, the water particles are fine, directly penetrate the muscle bottom, 16 times the effect of ordinary spray. Atomizing products, promoting absorption, replenishing water and injecting oxygen.

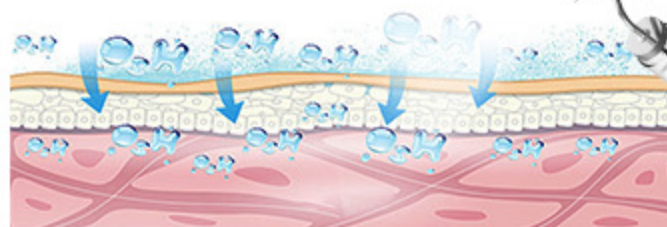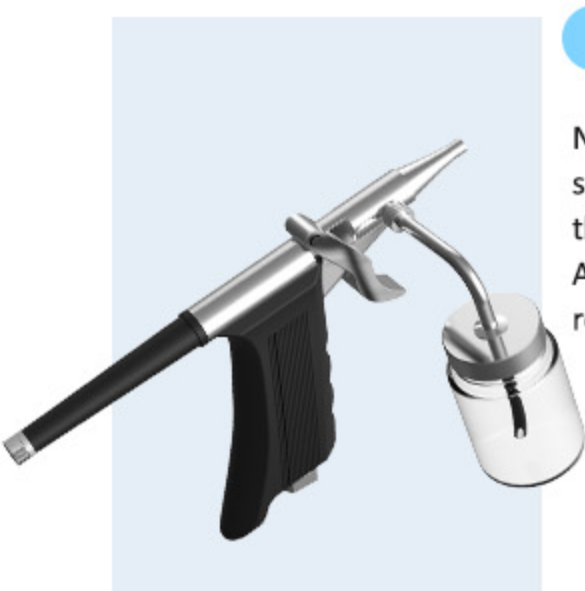


6 PLASMA

Using low-temperature plasma technology, the release of low-temperature plasma improves skin barrier function, suppresses inflammatory responses, and promotes collagen regeneration. Functions: Anti-bacterial and Acne-removing, Reduce acne scars, Anti-aging and Firming, Reduces Wrinkles, Fades Pigmentation.

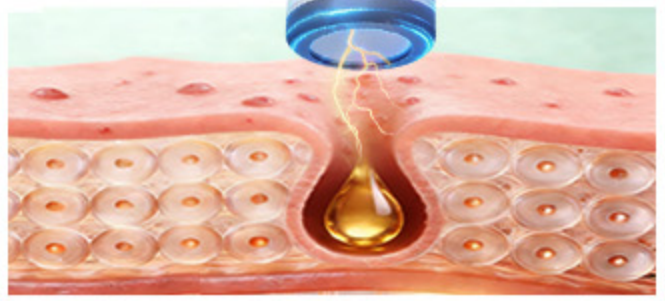

7 HIGH FREQUENCY

Generates high frequency current through the glass electrode tube, promotes blood circulation and promotes metabolism, Acne removal, skin care.

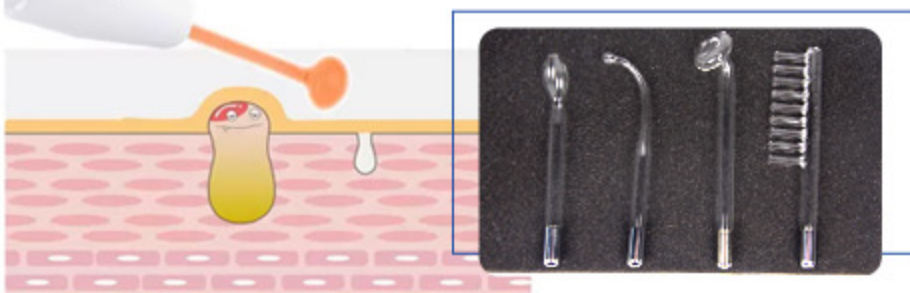


8 SKIN SCRUBBER

High-frequency vibration removes dead skin and blackheads to clean the face and tighten the skin. Can be used with gels, water emulsions, solutions, etc.




9 ICE/HOT HAMMER

Resolves pain and discomfort after skin management, calms and repairs skin and promotes product absorption.

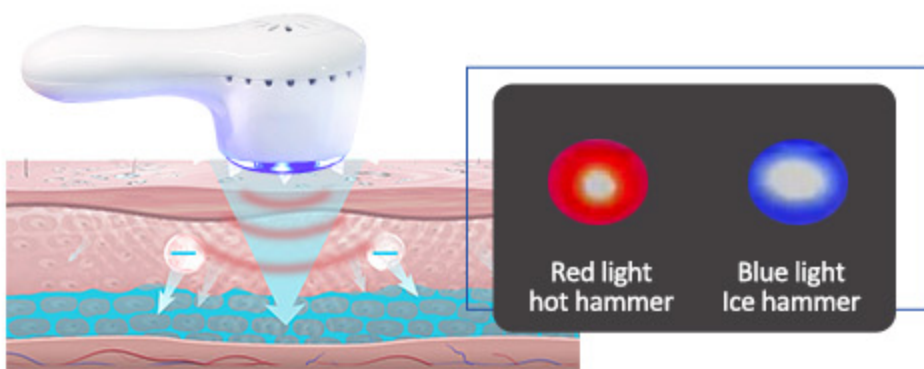


10 DR.PEN

12 microneedles vibrate at high frequencies, opening up skin channels for effective absorption and stimulating subcutaneous hair follicle tissue regeneration, accelerating rapid hair root growth

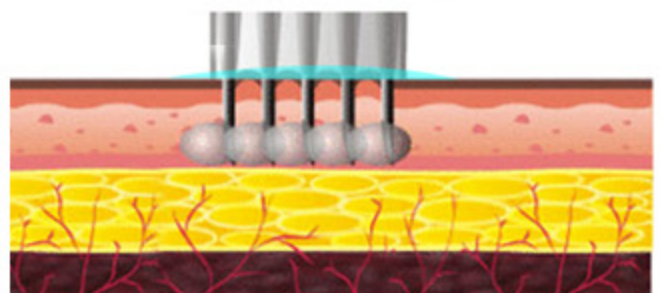

11 BIO HANDLE

Microcurrent is a low level of electricity that mirrors the body's own natural electrical impulses. these tiny microcurrent impulse encourage your body's currents at a cellular level. The signs of aging are greatly reduced, while skin tone and elasticity are dramatically improved.

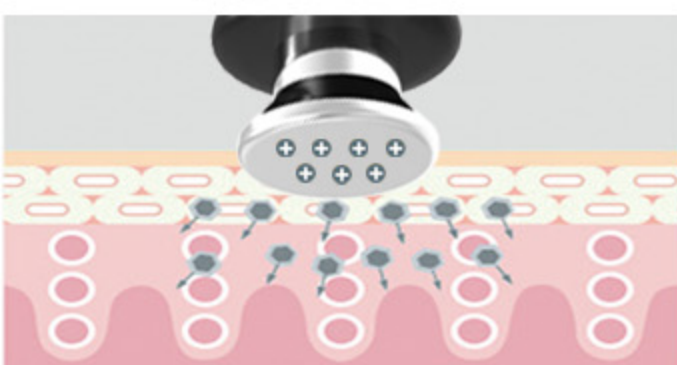



Product Parameter

Power: <150W

RF: 1MHz

Vacuum: ≤ 90 Kpa

BIO: 500Hz

High Frequency: 10W

Plasma Frequency: 60K

Screen: 15 inch Touch screen

Dr. Pen:

Needle Length : 0.25 – 3mm

Speed Control : 6 levels

Max Speed : 15000 times/min

Needle Number : 12 needles

Handles:

Hydrodermabrasion with 8 tips,

Diamond dermabrasion with 9

tips, Bi-polar / Three-polar RF,
High frequency, Skin scrubber,

Ice / hot hammer, Spray gun,

Dr.pen, Plasma, 2 BIO handle



Package Size: 118cm*55cm*47cm

N. W./ G. W.: 28kg/42kg

Voltage: 110V/220V, 50Hz/60Hz



Parts Introduction

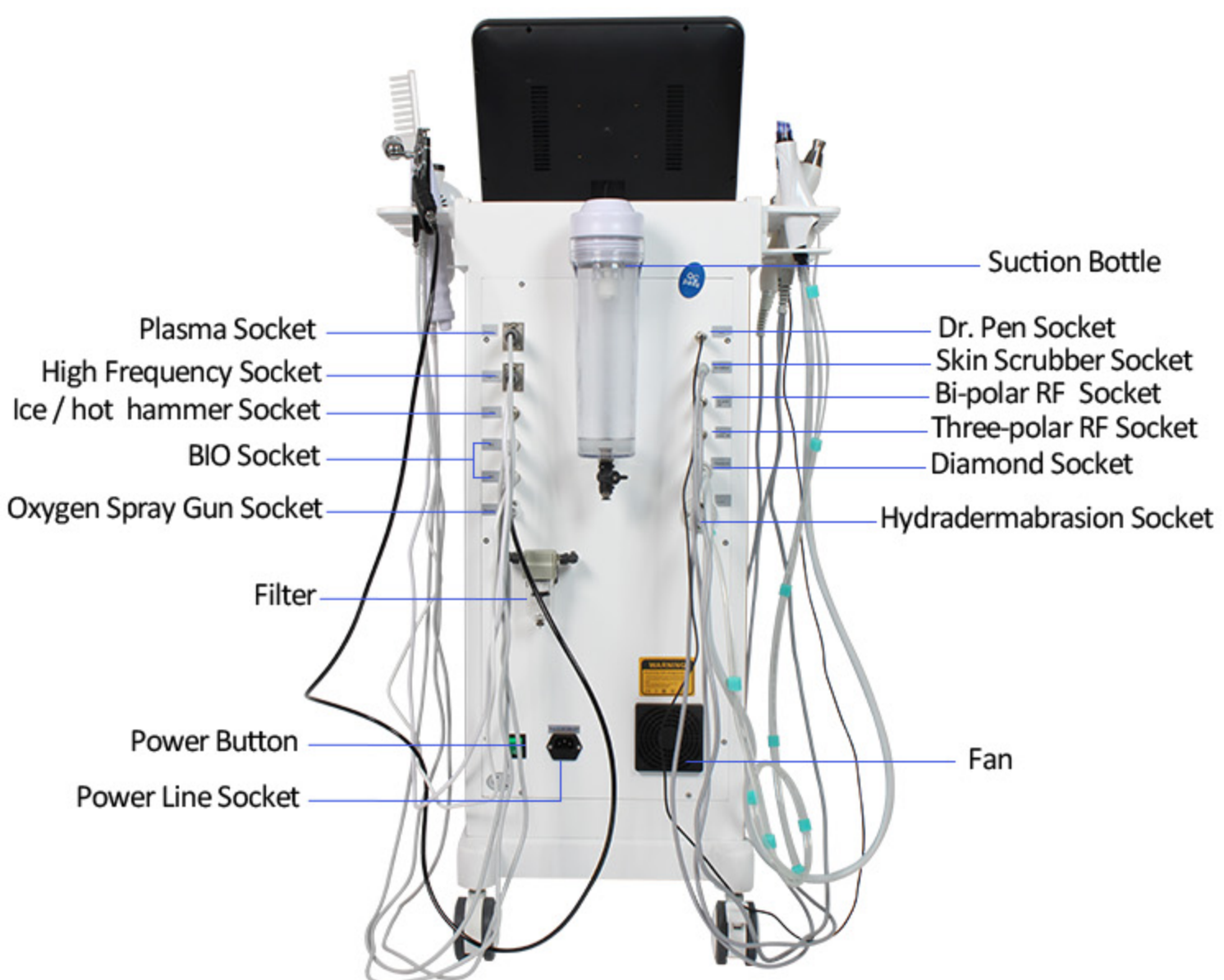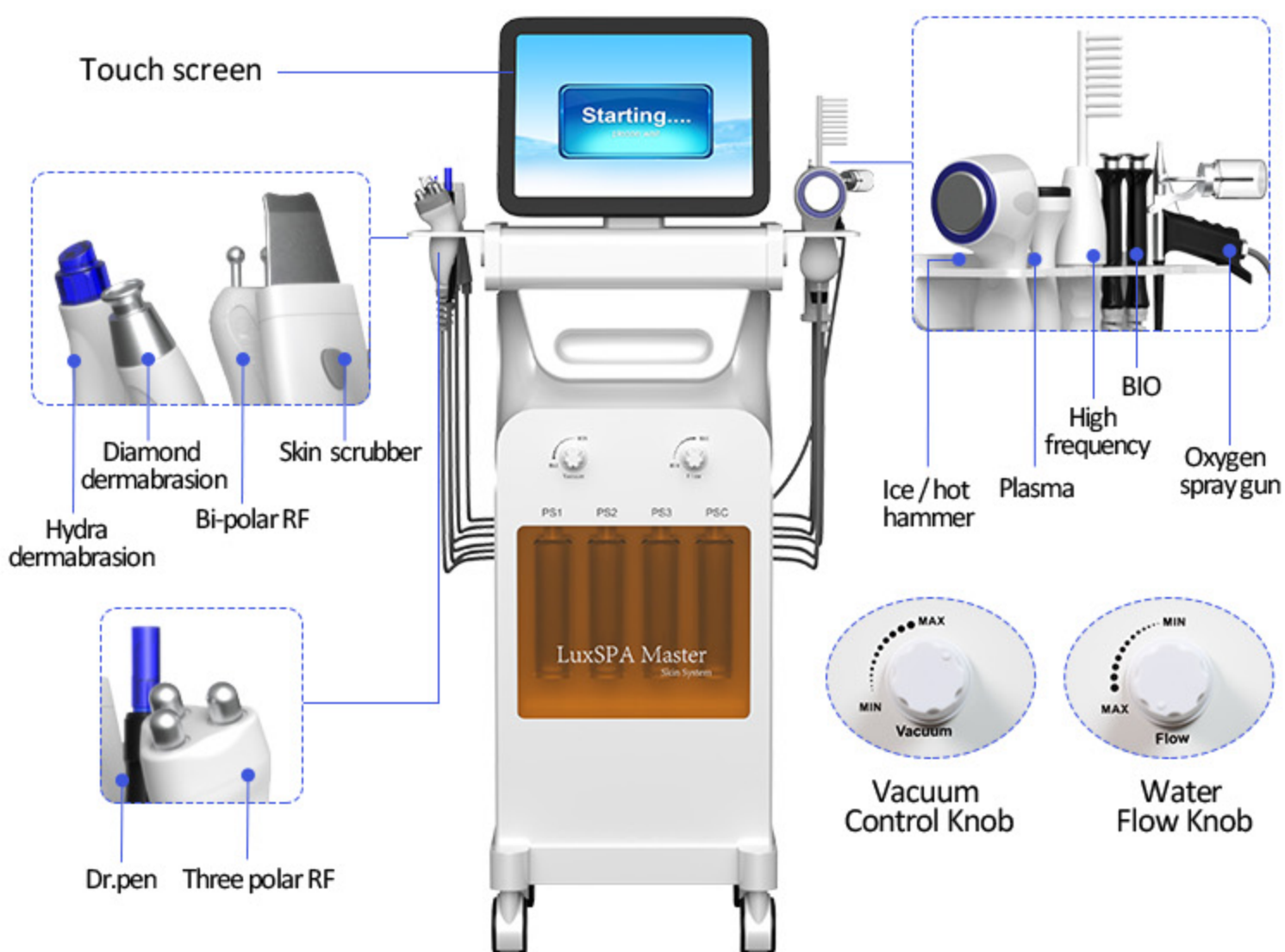

15-inch bigger touch screen







Easy operation



High Definition
LED Screen

Starting....
please wait

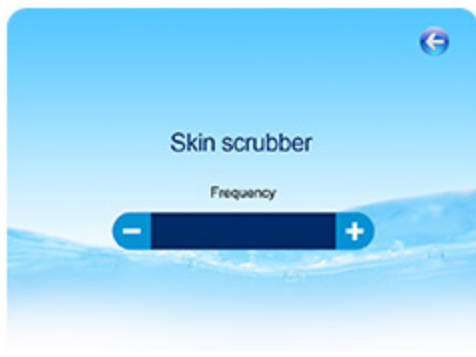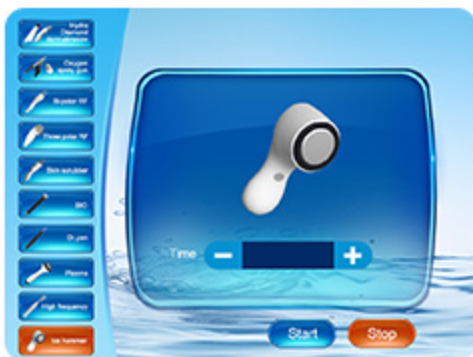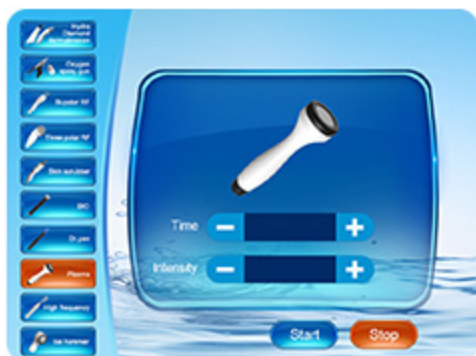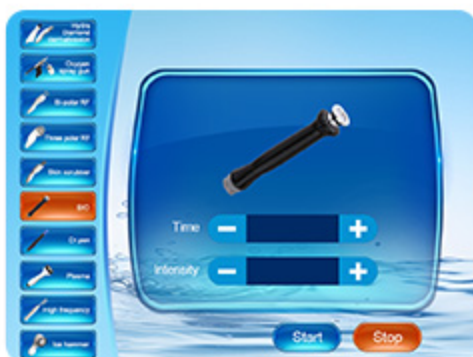

Hydra / Diamond Dermabrasion with tips

Microdermabrasion

Remove skin keratin

Skin rejuvenation



Oxygen Spray



Quick hydration,
smooth skin

Improves skin texture

Reduces excessive
keratin, purifies skin

lightening stains,
improve chlorosis

Skin Scrubber

Clearing blackheads without
damaging the skin

Lift and shape skin

Ultrasonic blackhead
removal

Remove the squamous
layer on the surface
of the skin



High Frequency

High frequency micro current

Pore shrinking



Acne treatment



Skin rejuvenation



Scalp unblocking





Bi-polar RF




Reduce fine lines



Delaying skin aging



Remove dark circles under the eyes

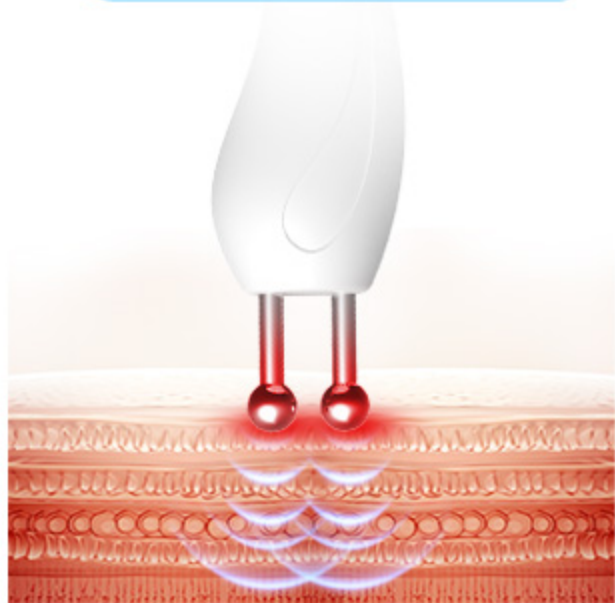

Three polar RF




Smooth wrinkles



Tighten the skin



Delay skin aging

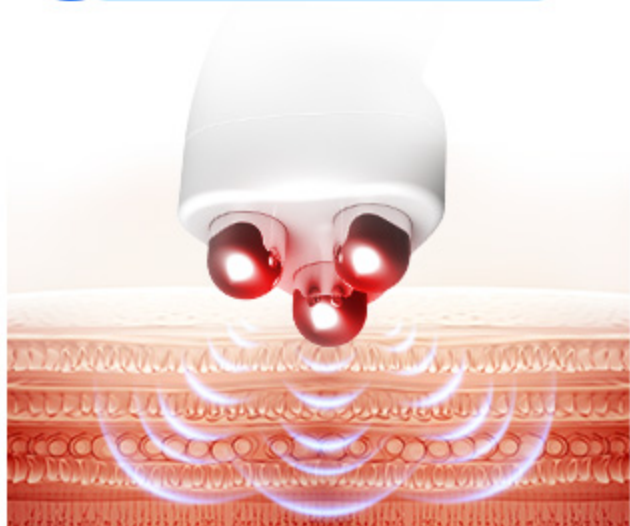

Ice/Hot Hammer Handle

Calms and repairs skin

Promotes product absorption

Press the button to switch between hot and cold hammers



Hot hammer





Ice hammer



Dr.pen Handle

12 microneedles vibrate at high frequencies, opening up skin channels for effective absorption and stimulating subcutaneous hair follicle tissue regeneration, accelerating rapid hair root growth

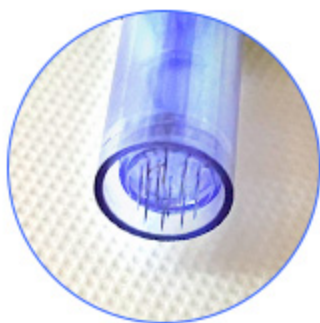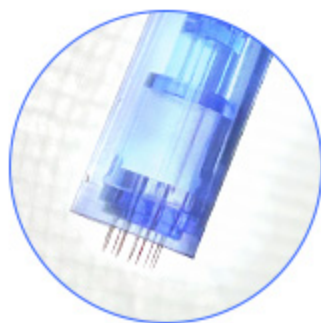


12 needles 0.25 – 3mm (Adjustable)

Rotate and adjust the length of the needle



6 Adjustable Strength

Two BIO handles

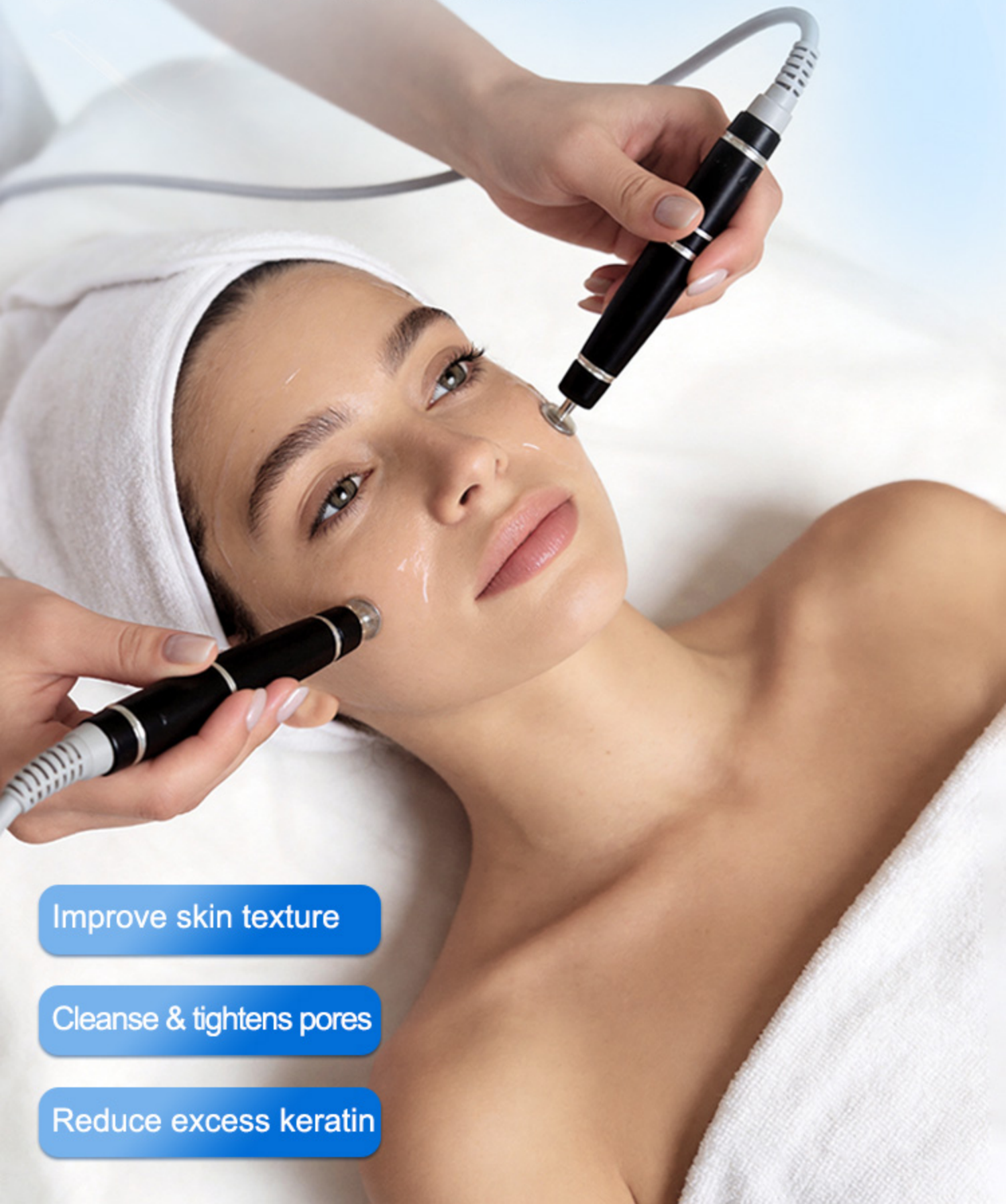

Improve skin texture

Cleanse & tightens pores

Reduce excess keratin

Plasma handle



Improve skin pigmentation

Fades Scars (especially acne pits)


Anti-bacterial and Acne-removing

Anti-aging and firming, reduces wrinkles, improving skin laxity

LuxSPA Master Function



Facial rejuvenation

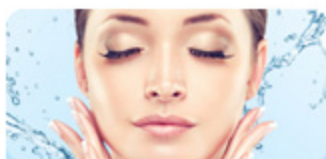

Skin hydration



Skin tightening



Remove acne



Wrinkle removal



Remove dark circles



Cleanse skin



Reduce blackheads




Delay aging



Soothing & calming skin



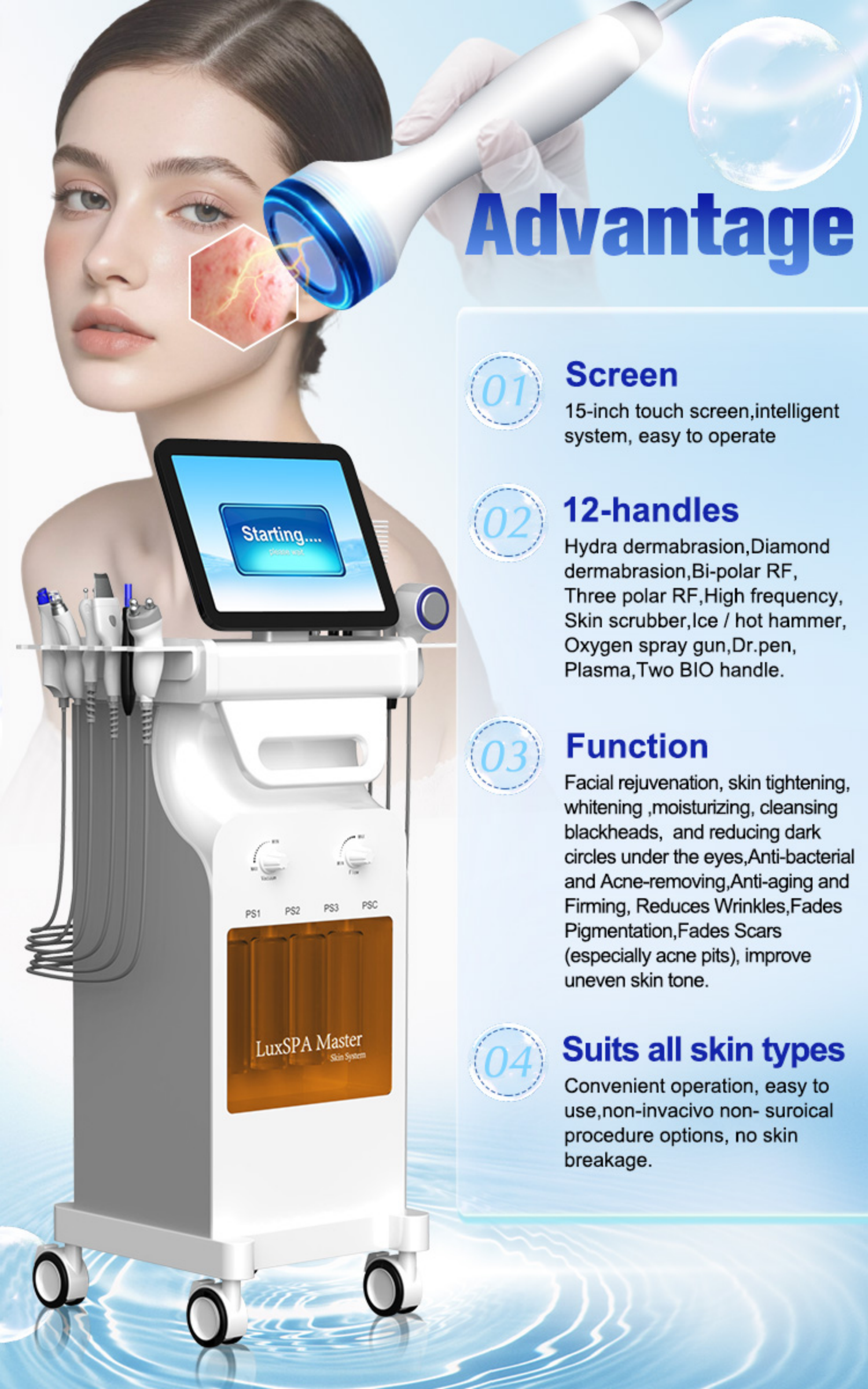

Advantage

01

Screen

15-inch touch screen,intelligent system, easy to operate

02

12-handles

Hydra dermabrasion,Diamond dermabrasion,Bi-polar RF, Three polar RF,High frequency, Skin scrubber,Ice / hot hammer, Oxygen spray gun,Dr.pen, Plasma,Two BIO handle.

03

Function

Facial rejuvenation, skin tightening, whitening ,moisturizing, cleansing blackheads, and reducing dark circles under the eyes,Anti-bacterial and Acne-removing,Anti-aging and Firming, Reduces Wrinkles,Fades Pigmentation,Fades Scars (especially acne pits), improve uneven skin tone.

04

Suits all skin types

Convenient operation, easy to use,non-invacivo non- suroical procedure options, no skin breakage.





Instrument Use Case



Real Picture



Before and After

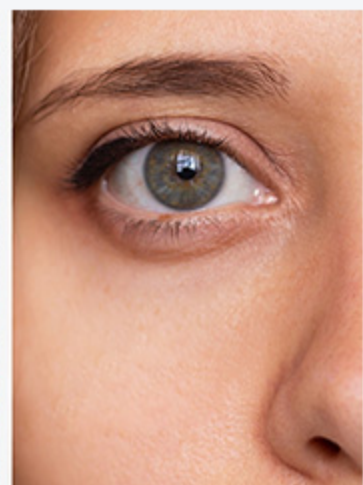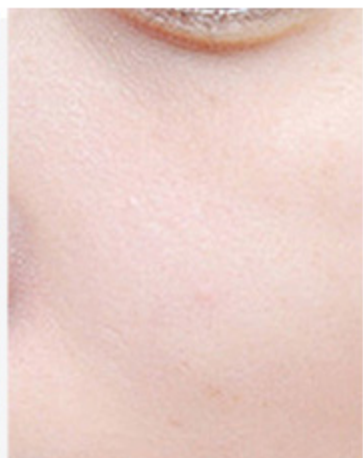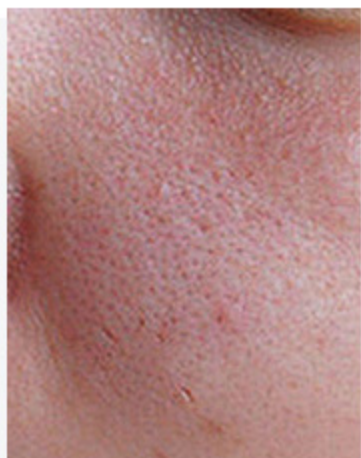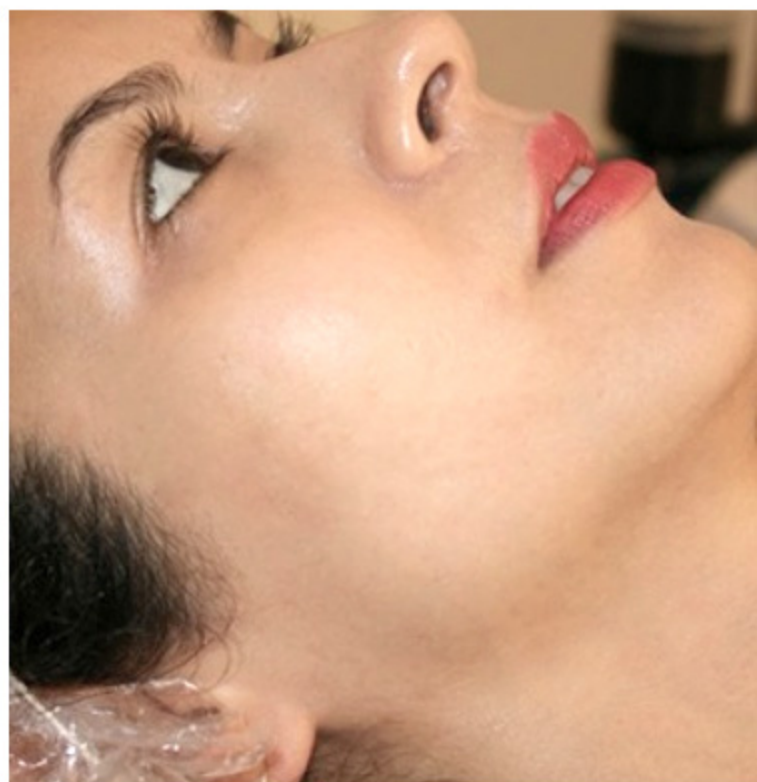

Operation Method





Neck (20-30minutes)

In the direction of the platysma, operate from the upper edge of the clavicle to the upper edge of the mandible. Avoid the laryngeal knots and operate carefully at the carotid artery, vein, and nerve; Point-like operation for neck lines; avoid laryngeal knots, carotid arteries, veins, nerves.

Cheek (20-30minutes)


The direction of the labial muscles starts from the mandible, avoids the chin and lifts to the corner of the mouth.

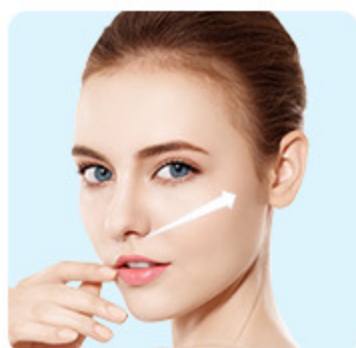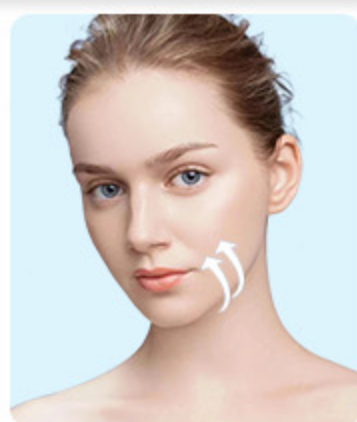


Zygomatic minor muscle direction. Operate from the nasolabial fold along the zygomatic minor muscle to above the cheekbones, avoiding the temples.



Orbicularis direction. Lift the corner of the mouth from the chin along the orbicularis oris muscle to the nasolabial fold, and then lift it up after the nasolabial fold.

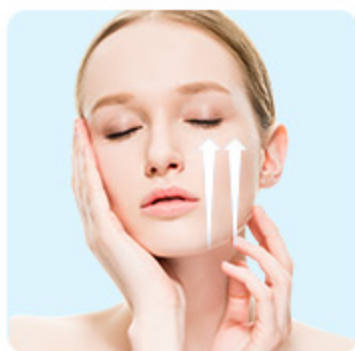

Masseter direction. Operate from the upper edge of the mandible along the masseter muscle to the cheekbone. (Note: It is strictly forbidden to operate on the nasal bone).

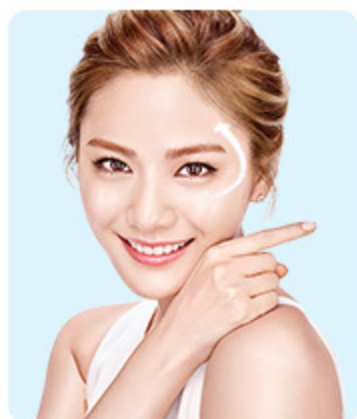

Eyes (10-15minutes)

Orbicularis oculi muscle direction. Starting at the 1/3 of the inner corner of the eye from the inferior orbit, raise the corner of the eye and the end of the eyebrow to the Orbicularis oculi muscle direction. Start at the middle of the inferior orbit, avoiding the eyeballs and temples, lifting the corners of the eyes, the end of the eyebrows to the hairline.

Forehead (10-15minutes)

Along the upper eye socket, from the brow, the brow waist, and the brow tail to the middle of the forehead. Raise from the upper orbit along the frontal muscle to the hairline.



Handles

Hydra dermabrasion



Diamond dermabrasion



Bi-polar RF



Three polar RF



High frequency



Skin scrubber



Ice / hot hammer



Plasma



Oxygen spray gun



Dr.pen



BIO




BIO



12 in 1 LuxSPA Master

Multi-functional spa
facial skin care

Quickly treat skin, Safe, comfortable, effective

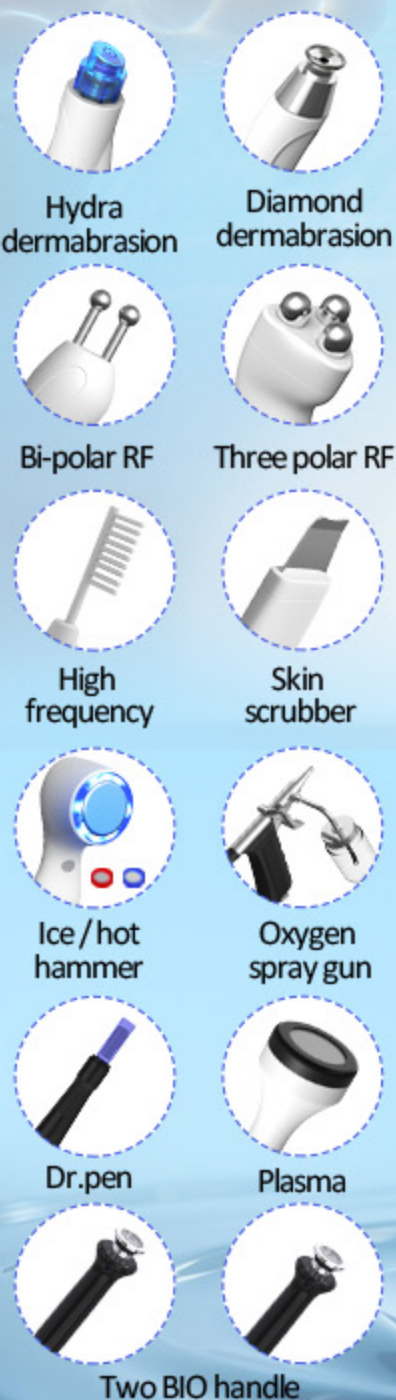

Product Function



Facial rejuvenation



Skin tightening



Wrinkle removal



Cleanse skin



Skin hydration



Remove acne



Remove dark circles



Reduce blackheads

Product Advantage

- 15-inch touch screen, intelligent system, easy to operate
- 12-handles: Hydra dermabrasion, Diamond dermabrasion, Bi-polar RF, Three polar RF, High frequency, Skin scrubber, Ice / hot hammer, Oxygen spray gun, Dr. pen, Plasma, Two BIO handle.
- Function: Facial rejuvenation, skin tightening, whitening, moisturizing, cleansing blackheads, and reducing dark circles under the eyes, Anti-bacterial and Acne-removing, Anti-aging and Firming, Reduces Wrinkles, Fades Pigmentation, Fades Scars, improve uneven skin tone.
- Convenient operation, easy to use, non-invasive non-surgical procedure options, no skin breakage.

Before and After

